

MAKING A GOOD IMPRESSION

Impression Mistakes to Avoid



Top of concha all the way to the helix is voided



Entire valley voided



Middle to top of concha under-filled



Concha and back of concha voided



From antitragus all the way to the helix is voided



Above antitragus voided



Concha from 2/3rd to helix voided



Canal not true to ear



Concha voided from 2/3rd shell to helix



Helix voided



Antitragus voided



Above antitragus voided and the entire back of concha voided



Above antitragus to top of concha voided



Heel voided due to hair



From half shell to helix is voided



From half shell to helix is voided



Concha voided



Incomplete Impression



Incomplete Impression



Incomplete Impression



Concha voided from 2/3rd shell to helix



Canal is not true to ear



Material did not set



Antitragus voided



Helix voided