

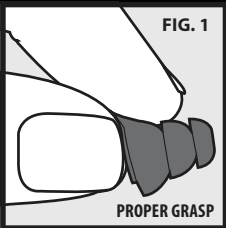


HOW TO INSERT/REMOVE TRU™ HEARING PROTECTION

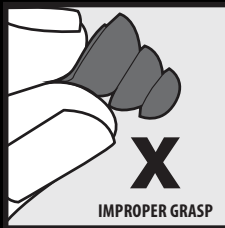

Grasp earplug by the tab, so that thumb is placed on surface of filter (Fig. 1). This will ensure that tab is easy to grasp for removal once inserted. Pull the top of your ear upward and outward and carefully insert your earplug until the eartip seals in the ear canal (Fig. 2). When removing, pull gently to break the seal and avoid discomfort.

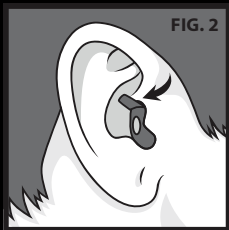

PROPER FIT: Notice that removal tab sits on top of earplug, and **not** trapped in the lower part of the ear opening (Fig. 2). This ensures that the tab is easy to grasp when removing the earplug.

OTHER WESTONE PRODUCTS



W SERIES



UM PRO SERIES



ADVENTURE SERIES



ELITE SERIES

Westone[®]

2235 Executive Circle, Colorado Springs, CO 80906

westone.com | 800.525.5071

© Westone Laboratories, Inc. | PN 98292 | rev 1