



## HOW TO INSERT/REMOVE TRU™ HEARING PROTECTION

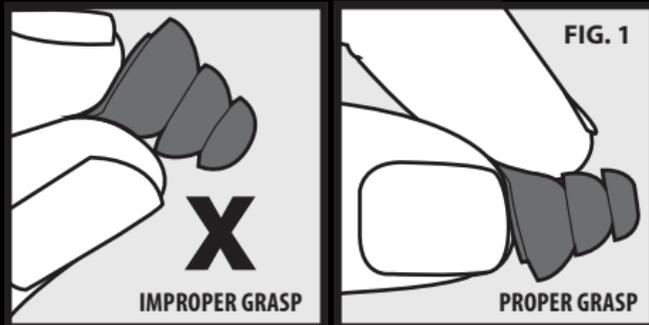

Grasp earplug by the tab, so that thumb is placed on surface of filter (Fig. 1). This will ensure that tab is easy to grasp for removal once inserted. Pull the top of your ear upward and outward and carefully push and insert your earplug until the eartip seals in the ear canal (Fig. 2). When removing, pull gently to break the seal and avoid discomfort.

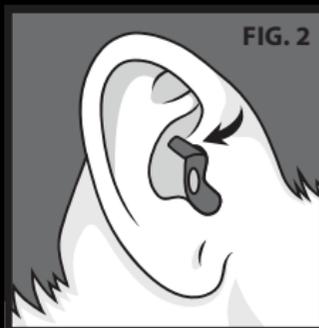

**PROPER FIT:** Notice that removal tab sits on top of earplug, and **not** trapped in the lower part of the ear opening (Fig. 2). This ensures that the tab is easy to grasp when removing the earplug.


# Westone®

*Westone has over fifty years of unparalleled knowledge and experience in the field of hearing. With hearing care and music specialists on our research and production teams, Westone is uniquely qualified to provide the best performing, best fitting, highest quality products. Over fifty years ago, we decided that "The Golden Rule" was the only way to do business. We still follow this simple guiding principle and strive to "do the right thing" in all of our actions. We're here to be of service and exceed your expectations.*

2235 Executive Circle, Colorado Springs, CO 80906  
westone.com | 800.525.5071

© Westone Laboratories, Inc. | PN 98297 | rev 1

# TRU™

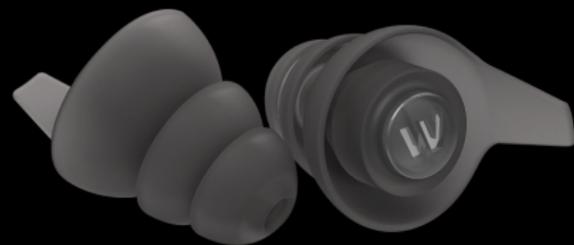

FULL-FREQUENCY UNIVERSAL-FIT HEARING PROTECTION


## OWNER'S MANUAL

## TRU™ FULL-FREQUENCY HEARING PROTECTION

- Full Frequency Hearing Protection
- Low-Profile Comfort Fit Silicone

Utilizing advanced acoustic filter technology, TRU hearing protection has been critically tuned to reduce volume yet maintain sound quality across the sound frequency spectrum. The result is a comfortable and transparent listening experience.

### HOW TO INSERT TRU™ FILTERS

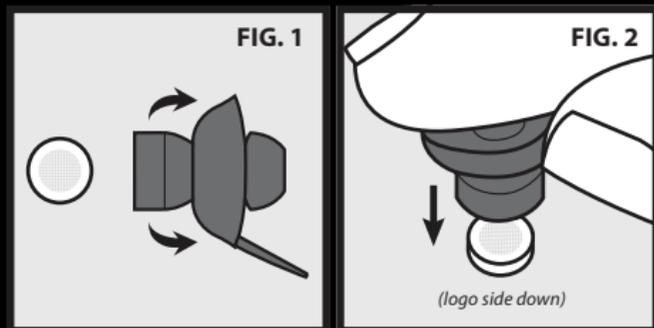

Place filter facedown on a flat surface. Fold back largest flange on earplug (Fig. 1). Place open end of earplug over filter and press down until filter snaps into filter pocket in earplug (Fig. 2). Logo should face out (Fig. 3)


## HOW TO CLEAN TRU™ HEARING PROTECTION

Clean your earplugs after every use. When needed, wipe clean with a damp cloth. Only wash earplugs after you have removed filters. Do not clean with harsh chemicals. Alcohol is not recommended.

## HOW TO STORE TRU™ HEARING PROTECTION

Store your earplugs in their protective case after every use. Do not store anything else in your earplug case.

### HOW TO REMOVE TRU™ FILTERS

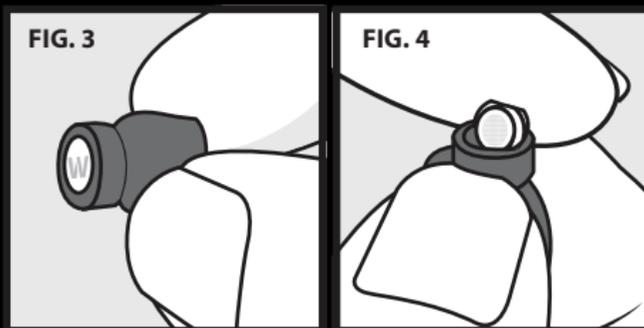

Fold back largest flange on earplug (Fig. 1). Place thumbnail at base of filter pocket and gently press into earplug until filter is forced out of filter pocket. Capture filter with opposite hand (Fig. 4).

## TIPS & WARNINGS

- Your earplugs comply with EN-352-2 and ANSI S 3.19 standards of measurement.
- To achieve the ideal hearing protection and attenuation, your earplug must be properly fitted, adjusted, and maintained. It must seal the ear completely. If unsure of size, start with the larger earplug.
- When exposed to potentially dangerous noise levels, use your earplugs continuously.
- Make sure your earplugs provide adequate protection for the noise level in your environment.
- To assure ongoing serviceability, regularly inspect your earplugs.
- Never remove filter while using your earplugs.
- Your earplugs may be adversely affected by certain chemical substances. Do not clean with harsh chemicals or alcohol.

**Hearing protection may be seriously compromised if warnings are not followed.**